USER MANUAL

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Shower Chair BUTTERFLY and CLASSIC
Duschhocker BUTTERFLY und CLASSIC

PLEASE READ THIS MANUAL CAREFULLY TO ENSURE SAFE USAGE AND MAINTENANCE

INTRODUCTION

Thank you for deciding to purchase one of our shower chairs. When used correctly it is designed to give many years of reliable trouble-free service.

PLEASE NOTE:

This equipment must be installed by a competent person.
136 kg (300 lbs) weight limit.

Do not exceed the weight limit stated. Doing so could put the user at risk.

BEFORE USE

1. Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the product.
2. Check the product for any visible damage. If you see any damage or suspect a fault, do not use the product, but contact your supplier immediately for support.
3. If all parts are okay, please dispose the packaging according to the local waste guidelines.

PART LIST

CLASSIC / BUTTERFLY

- 1 x seat board
- 5 x washer
- 5 x screw
- 1 x frame A
- 1 x frame B
- 4 x legs

SPECIFICATIONEN

CLASSIC / BUTTERFLY

- Length: 53 cm
- Height: 46 - 56 cm
- Width: 44 cm

Frame Material: Aluminium + PE HD

Weight Capacity: 136 kg

ASSEMBLY

TOOL-FREE ASSEMBLY. YOU DO NOT NEED A SCREWDRIVER. PLEASE MAKE SURE THE SCREWS ARE FIRMLY FIXED!

Lay the seat board flat on the floor so that the seating area faces the floor.

Identify frame A. (When stood up frame A has a flat top and a cut-out on the underside of the tube).

Attach frame A to the seat board, using 2 x screws and 2 x washers.

Identify frame B. (When stood up frame B has a cut-out on the top of the tube and the underside of the tube is flat).

Attach frame B to the seat board, using 2 x screws and 2 x washers.

Secure both frames to the seat board using 1 x screw and 1 x washer.

Take one of the legs and put it onto one of the frame ends. Push down the securing pin and twist the leg to the side to allow you to slide the leg up or down to the desired height.

If you found the correct height, rotate the leg back to allow the securing pin to pop out of the hole and fix the height at the correct setting. Make sure the securing pin popped out completely.

Repeat steps 7 and 8 for each leg, ensuring that all four legs are set to the same height setting.

HEIGHT ADJUSTMENT

For adjusting the height, please look at ASSEMBLY point 7-9.

CLEANING

All of the product components are manufactured from quality materials. However, it is very important that you only clean your shower chairs and stools using a non-abrasive cleaner or mild detergent with a soft cloth, avoid using lime scale remover.

WARNING! Abrasive cleaners e.g. AJAX and or abrasive cleaning pads will seriously damage your product beyond repair. Do not exceed 50°C when cleaning the seat as this will permanently damage the plastic.

NOTE: Regular daily cleaning will reduce the buildup of limescale on your product.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety. This includes:

- Bent or damaged tubes or cracks at the tube joints.
- Elongated adjustment holes.
- Loose or worn fasteners
- Smooth, worn or split ferrules (rubber tips).

REPLACE ALL WORN, MISSING OR DAMAGED PARTS WHERE NECESSARY. IF IN ANY DOUBT, PLEASE DO NOT ISSUE OR USE, BUT IMMEDIATELY CONTACT YOUR SUPPLIER FOR SERVICE SUPPORT!

CARE & MAINTENANCE

Please make a check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by DMEone or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

LIMITED LIFETIME WARRANTY

- The shower chairs have a limited lifetime warranty to be free of manufacturing defects in materials and workmanship under normal applications.
- If this product becomes inoperable due to defect and requires repair, return the product with all component pieces and proof of purchase to your local dealer.
- This warranty does not cover any shipping/transport costs.
- This warranty does not apply if the product is subject to misuse, neglect, rough handling or damage.

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