USER MANUAL

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STANDARD Rollator Aluminium (foldable)

STANDARD Rollator Aluminium (faltbar)

x **DME**one

PLEASE READ THIS MANUAL CAREFULLY TO ENSURE SAFE USAGE AND MAINTENANCE

INTRODUCTION

The DMEone Aluminium Foldable Rollator STANDARD is equipped with a seat, backrest and bag, and features a robust, foldable frame, adjustable handle height and adjustable seat height. It has 4 robust wheels for added stability and a lightweight frame for easy transport and manoeuvrability. The rollator is intended solely as a walking aid for people with restricted mobility. It is supposed to support people who have balance problems or other walking restrictions and is supposed to give them stability and security while walking.

INTENDED USE

The rollator is

- intended for private use.
- designed for indoor and outdoor use.
- NOT suitable for sand, gravel, very uneven terrain and very steep terrain.
- a medical product and conforms to the EU directive 2017/745 about medical devices. It is NOT a toy.
- only designed for persons up to 136 kg (300 lbs).

Use the rollator only as described in the instructions for use. Any other use is considered improper and can lead to material damage or even damage to persons. The manufacturer or dealer assumes no liability for damage caused by improper, incorrect use.

PLEASE NOTE:

This equipment must be installed by a competent person.

136 kg (300 lbs) weight limit. Do not exceed the weight limit stated.

Doing so could put the user at risk.

PART LIST

1 x frame	1 x bag
1 x backrest	4 x screws for frame
2 x rear wheels with handle bar	2 x screws for handle bar
2 x front wheels	2 x washers

BEFORE USE

1. Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the product / the surface of the product.
2. Check the product for any visible damage. If you notice any damage or suspect a fault, do not use the product, but contact your supplier immediately for support.
3. If all parts are okay, please dispose the packaging according to the local waste guidelines.

DELIVERY SCOPE OF STANDARD ROLLATOR

1. Frame with seat
2. Rear wheels with handles
3. Front wheels
4. Bag
5. Screws
6. Back rest

ASSEMBLING THE ROLLATOR

1. Take the frame (no. 1, picture above) and unfold it.
2. Take the front wheels (no. 3, picture above) and insert them at the front part of the frame (see fig. 1, no. 3). Ensure that the securing pin of the wheel frame pops out and secures the wheel on the frame.
3. Take the rear wheels and the handle bars (no. 2). Insert the tube section of the wheels into the bottom of the frame and the tube section of the handle bars into the top (fig. 1, no. 2). Please take care of the brake cable. Place it on the outside and clip it with the cable holder onto the frame and adjust the silver screw of the brake cable to activate the brake (fig. 2). Make sure the securing pin of the wheel tube pops out and secures the wheels.

4. Make sure the hole on the frame is aligned with the hole on the handle bar at the desired arm height. Insert one of the bolts (fig. 3) so that the bolt-head is pointing away from the rollator. Tighten the bolt. Repeat for the opposite side. For correct handle height see corresponding chapter.

fig. 2

fig. 3

fig. 4

5. Take the four screws that look the same and place them into the frame shortly above the wheels and fasten them (fig. 4).
6. Insert the back rest into the provided holes (fig. 5) until you here a „click“ sound. Now it is fixed.
7. Take the seat and lift it up. Now, take the bag and fasten it with the hook and loop fastener around the cross bars underneath the seat.

fig. 5

CORRECT HANDLE HEIGHT


Stand straight and close to the rollator. Let your arms hang loosely. Now grasp the handlebars of the rollator. If the arms are slightly bent, the height of the handles is adjusted correctly. If this is not the case, please adjust the height of the handlebars as described in the next paragraph.

HANDLE HEIGHT ADJUSTMENT

The height level of the handles can be adjusted to different heights.

1. Please loosen the screw (fig. 6) which fastens the handle bar.
2. Push down or pull up the handle to the height you need.
3. Make sure the hole on the frame is aligned with the hole on the handle bar. Insert the bolt back into the hole and tighten.
4. Repeat the process for the second handle.

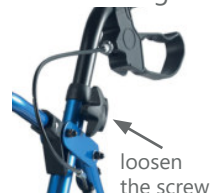fig. 6

NOTE: The handles are set at the correct height when the handles are at the crease of your wrist while your arms hang loosely.

SEAT HEIGHT ADJUSTMENT

The seat height can be adjusted to the users size. It is at the correct height when the user sits on the seat and the upper and the lower leg form a 90° angle.

fig. 7

fig. 8

1. Please loosen the screws above the wheels (fig. 7).
2. Press the securing pin (fig. 8) and push or pull the tube of the castor until the securing pin adjusts to the next height.
3. Repeat step 2 until the desired height is reached. Make sure the securing pin pops out completely.
4. Repeat the above listed steps for the other 3 wheels.
5. If the desired height is adjusted, insert all screws and fasten them!

FOLDING THE ROLLATOR

To fold the rollator, remove the bag, than flip the locking mechanism and press the legs of the rollator together.

DANGER OF INJURY!

Keep hands away from the side braces when folding the rollator.

USING THE ROLLATOR

GENERAL INFORMATION ON USING THE ROLLATOR

It is essential to observe the following information on using the rollator. These instructions, the safety instructions and the intended use must be strictly observed for safe use!

**DANGER OF INJURY!**

- Check the condition of the rollator regularly. DO NOT use the rollator, if you notice any external damage.
- Always check that all screws are tight before using the rollator.
- Before using the rollator always check that the brakes are working properly.
- The rollator is a walking aid. Only use the rollator to be safe while walking. DO NOT use the rollator as a wheelchair or similar.
- Rain, moisture and dirt can affect the rollator's braking performance negatively.
- Avoid unsuitable surfaces and paths. Do not drive the rollator over gravel, sand or similar, uneven surfaces.
- On sloping paths, the rolling speed of the rollator can be higher than your walking speed. There is a risk of falling! Brake sufficiently and walk carefully.

WALKING CORRECTLY WITH THE ROLLATOR**DANGER OF INJURY!**

- The rollator must always be held close to the body while walking so that it provides good support and enables safe walking. If the rollator is too far away from the body while walking, it can easily roll away. The same is true if you walk too bent over. Danger of falling!
- When walking, support yourself on the rollator and stabilize your walking.
- Always walk upright and keep your eyes straight ahead.
- Only use the rollator if the handle height is adjusted correctly for your height (see CORRECT HANDLE HEIGHT).

USING THE BRAKES

Using the brakes while walking:

To reduce your walking speed, pull both handbrake levers towards the handle. Pull harder for greater braking effect. To reduce the braking effect, carefully release the handbrake lever.

Locking the brakes:

Press down on both handbrake levers until they click and stay in the down position. The brakes are now locked.

**DANGER OF INJURY!**

When stopping, parking and especially when you want to sit down on the rollator always lock both brakes! Check if both brakes are still locked before you get

up from the rollator in order to allow you to get up safely.

Untighten the parking brake:

Pull the handbrake levers up until they release and stop in the neutral position. The brakes are now released.

NOTE: The brakes on the rollator are factory set and should not require adjustment. If you think, the brakes are no longer working correctly, please contact your local dealer for technical assistance.

GETTING OVER OBSTACLES

DANGER OF INJURY!

- Hold the rollator firmly when getting over obstacles.
- Be especially careful when walking down curbs!
- Do not use the rollator on stairs!
- Do not tilt the rollator more than 5 degrees to the side. Risk of falling!

USING THE SEAT CORRECTLY

DANGER OF INJURY!

- Only sit on the rollator when both parking brakes are activated and the rollator is on a level surface!
- DO NOT allow yourself to be pushed when you are sitting on the rollator and do not push any other person sitting on the rollator. Danger of falling!
- Do not sit on the rollator if the surface is very smooth and slippery. The rollator can slip away even though the parking brake is activated.
- Only one person is allowed to sit on the seat at a time. Please pay attention to the maximum load capacity of the rollator!
- Always sit in the middle of the seat and make sure that both feet touch the floor!

CLEANING

Clean the rollator with a soft, damp cleaning rag and, if necessary, with a little mild detergent. Then dry the rollator well. Let the rollator dry for a while after cleaning so that hard-to-reach areas can dry thoroughly. In particular, the wheels, handles, and braking devices must be completely dry before using the rollator again.

DANGER OF DAMAGE!

Never use organic solvents and / or abrasive cleaner.

CARE & MAINTENANCE

Check the functionality of the rollator before each use. In particular, check the following points:

- The brakes are in perfect working order and the brake cables are free of deformations and tears.
- The wheels are in perfect working order. All wheels can be rotated properly and the front wheels can be pivoted. The tires do not show any significant signs of wear.
- All screw connections are tight.
- The entire rollator is free of damage, cracks and coarse soiling.

DANGER OF INJURY!

DO NOT use the rollator if the functionality of the rollator is not guaranteed or only guaranteed to a limited extent. Even minor damage or soiling (e.g. on the brakes of the rollator) can lead to injuries and accidents.

STORAGE

DANGER OF INJURY!

If you are not going to use the rollator for a longer period of time, secure the rollator with the parking brake against rolling away and only park the rollator where no one can fall over it.

- If the rollator is not going to be used for a longer period of time, clean it before you put it away (see CLEANING).
- Store the rollator in a dry and well-ventilated place.
- Protect the rollator from frost and direct sunlight.
- Keep the rollator out of the reach of children.

**DANGER OF DAMAGE!**

Never leave the rollator outside for a long period of time and do not expose the rollator to bad weather conditions. This can cause damage to the rollator.

TRANSPORTATION

Fold the rollator for transport (see FOLDING THE ROLLATOR). Make sure that the rollator is properly secured during transport. If necessary, take additional precautions to secure transport.

SECURITY ADVICE

Danger for children and people with reduced physical, sensory or mental abilities (e.g. partially handicapped people, elderly people with limited physical and mental abilities) or absence of experience and knowledge (e.g. children).

- People with reduced physical, sensory or mental abilities or lack of experience and knowledge may only use the rollator under supervision, or have been instructed in its safe use and have understood the dangers that can arise when using the rollator.
- Keep children away from the rollator. Children could tip over with the rollator and injure themselves.

**DANGER OF INJURY!**

Improper handling of the rollator can lead to serious injuries. It is essential to pay attention to the following information:

- DO NOT use the rollator if you are under the influence of substances that affect your mental and physical performance (e.g. strong medication or alcohol).
- DO NOT use the rollator on gravel, sand or similar unstable and uneven surfaces.
- DO NOT use the rollator on steep inclines.
- NEVER use the rollator on stairs, escalators or moving walkways.
- Use caution when operating rollator on wet, slippery or icy surfaces (e.g. ice and snow). It is best to NOT use the rollator on such surfaces.
- DO NOT walk with the rollator in the dark or in poor visibility conditions.
- Avoid driving the rollator over manhole covers, gratings, pans in the ground and the like. The tires could get caught in the bumps.
- Only walk with the rollator on the sidewalk. Use the roadway / street only for crossing.

- Never exceed the maximum load capacity of the rollator. The maximum user weight for the rollator is 136 kg.
- Never use the rollator as a wheelchair or as a means of transportation for people. This means: Do not push the rollator when someone is sitting on the rollator.
- DO NOT hang any bags on the rollator, except for the bag included in the delivery scope.
- DO NOT make any changes to the rollator, especially NOT to the moving parts of the rollator. If anything is changed, safe use is no longer guaranteed. You could injure yourself.
- Be sure to set both parking brakes before sitting on or setting down the rollator. The rollator is only suitable for short-term sitting and is by no means intended as a chair for permanent sitting. Before you get up from the rollator again, check that both parking brakes are still set. Only then get up.
- NEVER stand on the rollator. Never use the rollator as a climbing aid or as a ladder!

RISK OF BURNING!

Do not expose the rollator to direct sunlight or similar heat sources. Do not place the rollator in front of an open fire. Individual parts of the rollator can get very hot.

RISK OF DAMAGE!

Protect the rollator from permanent direct sunlight and other heat sources, as parts of the rollator can fade and become damaged over time.

CAUTION

Do not use this product without proper instruction from a healthcare professional.

- This product has a patient weight limitation of 136 kg.
- DMEone assumes no liability for any damage or injury caused by improper installation of this product.
- If components are missing or damaged, DO NOT USE. Contact your dealer immediately.
- Ensure all parts and components are tightened before use.
- Inspect rollator weekly for proper function and wear. If any component is not working or is worn, please contact your dealer immediately for replacement parts.
- If any serious incident occurs with the device it should be reported to the manufacturer and local authorized dealer.

WARRANTY

To improve the safety and lifespan of your walker rollator, it is recommended that you have it serviced on a regular basis. There is a comprehensive twelve-month warranty from the date on which your new walker rollator is delivered. The warranty covers the walker for repairs or replacement during this period. For more detail, please see the warranty conditions below:

1. Any work or replacement part installation must be carried out by an authorised dealer / service agent.
2. To apply the warranty should your walker rollator require attention please contact the outlet from which you purchased the walker.
3. Should any part of the walker rollator require repair or full or part replacement, as a result of a manufacturing or material defect within the warranty period, parts will be supplied free of charge. Note: The guarantee is not transferable.
4. Any repaired or replaced parts will be covered by the balance of the warranty period on the walker.
5. Parts replaced after the original warranty has expired will be covered by a three month warranty.
6. Consumable items supplied will not generally be covered during the normal warranty period unless such items require repair or replacement clearly as a direct result of a manufacturing or material defect. Such items include (among others): upholstery and tyres.
7. The above warranty conditions apply to brand new walkers. For whether your walker rollator is covered contact your dealer.
8. Under normal circumstances, no responsibility will be accepted where the walker rollator has required assistance as a direct result of:
 - a. the walker rollator part not having been maintained in accordance with the manufacturers recommendations.
 - b. failure to use the manufacturer's specified parts.
 - c. the walker rollator part having been damaged due to neglect, accident or improper use.
 - d. the walker rollator part having been altered from the manufacturer's specification or repairs having been attempted before the dealer is notified.

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.

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